

NEAL UNDERPAD



Neal Underpad is tubular bandage used as cast padding and manufactured from Aerosilver yarn so it is skin-irritate free and water-repellent. It drains water well and dries very fast to provide extraordinary comfort to patients. With Neal Underpad, patients can take a shower or even swim with limbs plastered.

Neal Underpad is Antibiotic.

Neal Underpad is woven with Aerosilver yarn blending quickdrying function and antibiotic functions together. So it is not only safe to patient's skin but also unpleasant odor free.

Neal Underpad is no skin irritated.

Even though it is water-repellent, it doesn't make any skin irritation and this is proved by skin irritation test(ISO 10993-10:2002(E)) by Korea Testing & Research Institute.

Neal Underpad provides with excellent comfort.


Neal Underpad acts as an excellent cast padding between skin and casting tape with its marvelous elasticity and softness.

Neal Underpad is easy to use.

Neal Underpad is tubular bandage, which does not need annoying bandaging work like when bandage and stockinette used.




Order Code

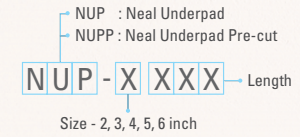


NUP-4100	Neal Underpad, 4inch x 10m	1 Roll / Box
NUPP-4140	Neal Underpad Pre-cut, 4inch x 140cm	10 Pcs / Box

How to use

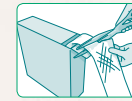

1. Cut Neal Underpad twice as long as application area.

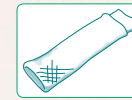


2. Fold it inside out to make it double layered.



3. Put it on the application area.



4. Fold and roll up the extra edge to make it neat.

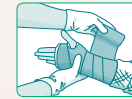

5. Make a cast with Neal Cast for the best result.

