

Why? Why toes can not be protected?



# Cast Shoe & Toe Guard

[PCT Pending : PCT/KR2013/005398] [Design Patent Registered : No. 30-0577372]  
 [Patent Registered : 10-1243588-000] [Patent Pending : No. 10-2013-0087009]



## Offer for Masterpiece Treatment!

**Different colors per size!** Enables to adjust the angle!  
 Compatible with the products of other company brands!

### Objective of Usage

A shoe equipped with a toe protection for a cast-treated patient

### Usage

1. Loosen the laces of the cast shoe and lift them up the toe guard.
2. A feet in a cast is put on the cast shoe and then the toe guard is lowered
3. Bond a shoelace and properly wear it.



[www.ggmed.com](http://www.ggmed.com)

