

QAL Medical OrthoAgility™


Made in the USA

6000X

The WaveFlex 6000X Hand CPM features a revolutionary design that is both lightweight and balanced, resulting in a durable and easy-to-use CPM device that allows patients to achieve a full composite fist at 270° at their own comfortable pace.

Benefits Overview

- Full composite fist flexion
- Lightweight and portable
- Comfortable, fully adjustable splint
- Adjustable speed
- Pause feature for enhanced results
- Programmable reverse-on-load and warm-up features
- Easy-to-use digital motion controller
- Minimal joint compression

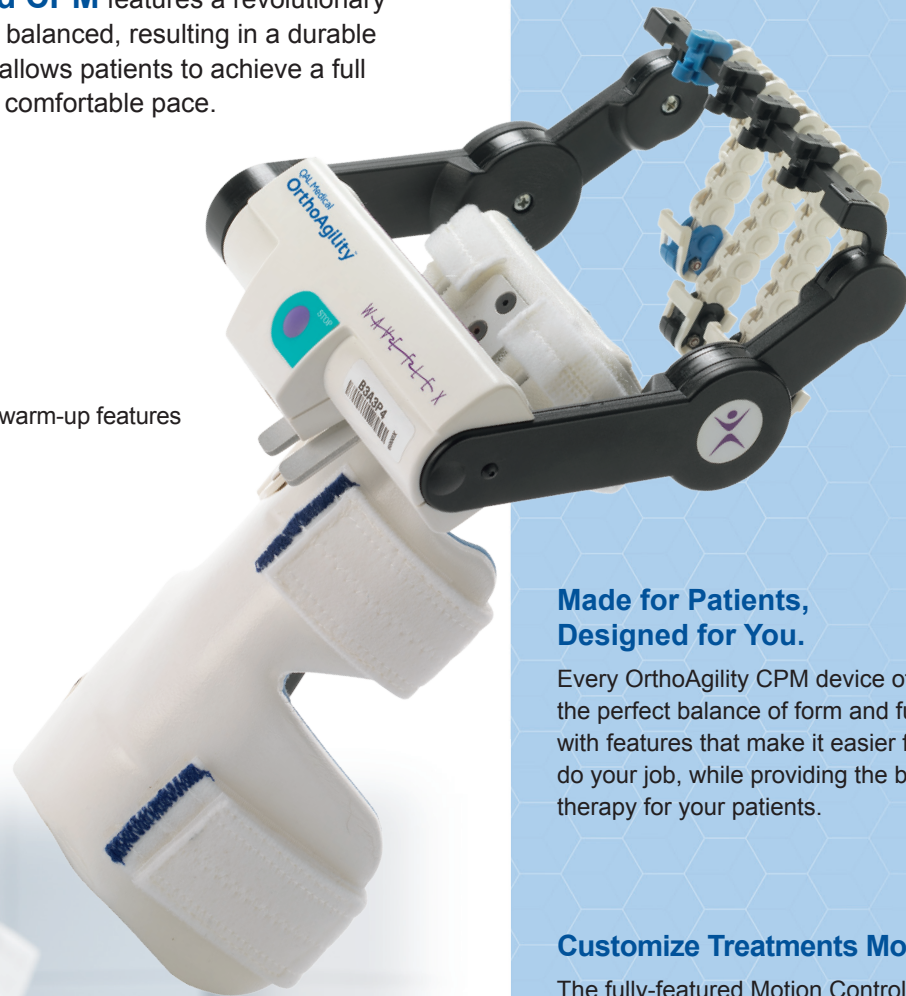

Made for Patients, Designed for You.

Every OrthoAgility CPM device offers you the perfect balance of form and function with features that make it easier for you to do your job, while providing the best in CPM therapy for your patients.

Customize Treatments More Easily

The fully-featured Motion Controller makes it especially easy to customize treatment to meet every patient's therapy needs.



Easily control Speed, Range of Motion, Pause, Warm Up feature and more with the convenient hand-held Motion Controller.





A True Original.

The OrthoAgility line of devices is the latest generation of products based on the original, global-first work of Dr. Robert Salter of the University of Toronto. We continue that tradition of originality by bringing fresh ideas about how to best integrate design and technology into products that are user-friendly for medical professionals, and provide the best of both the upper and lower body joint therapy for patients.

Ordering Information:

6000X	WaveFlex Hand CPM, patient kit, batteries, power supply
6301 (child)	Hand splint
6302 (adult)	Hand splint
6305	Finger clips (8)
6508, 6507, 6506	Finger actuators (small, medium, large)
FBA-006	Finger strips (attachment tape)
6304	VHB adhesive tape
6417	MP block kit
6511 (L), 6512 (R)	Glove small
6513 (L), 6514 (R)	Glove medium
6515 (L), 6516 (R)	Glove large

Specifications:

Range of Motion	MCP	0° - 90°
	PIP	0° - 110°
	DIP	0° - 70°
Full Composite Fist	270°	
Rate of Motion	30 - 120 seconds per cycle	
Pause (program)	0 - 30 seconds at end of flexion and/or extension	
Reversing Force	Low	1.6 kg (3.5 lbs)
	High	4.3 kg (9.5 lbs)

Warm-up (program)	Start at 50% of programmed range of motion	
Weight	Drive Assembly	0.45 kg (1 lb)
	Hand Controller	0.20 kg (0.45 lb)
	Patient Kit	0.21 kg (0.46 lb)
Power Requirements	Input	100 - 240 VAC, 50/60 Hz 40VA
	Output	5VDC, 2.5A
Power Source	Battery 6V-4 AA or use Power Supply H6K-001	
Low Battery Warning	L.E.D.	

Innovation in Motion

The 6000X, part of OrthoAgility's full line of CPM devices, features revolutionary full composite fist technology and an adjustable splint to ensure maximum patient comfort during rehabilitation, making your job easier and treatment more effective.



Indications Include:

Immediate postoperative management after the following, where indicated:

- Capsulotomy, arthrolysis and tenolysis for post-traumatic stiffness of MP & PIP joints
- ORIF of intra-articular, diaphyseal and metaphyseal fracture of the phalanges and metacarpals
- Flexor and extensor tendon tenolysis
- Flexor and extensor tendon synovectomies, post arthrotomy/drainage of acute septic arthritis
- Prosthetic replacement of MP and PIP joints
- Stable fractures
- Crush hand injuries w/o fractures or dislocations
- Hand or digit reattachment
- Dupuytren's contracture release
- Burn injuries; and reflex sympathy dystrophy (RSD)

Contraindications

Do not use the device if any of the following are present: untreated or uncontrolled infection, unstable fractures or hemorrhaging.

Note: Upon using the device, if signs of infection such as hyperthermia, fever, redness, irritation, warmth, swelling, bleeding, and/or increased persistent pain are present, discontinue operation of the device and contact the patient's physician. Do not proceed with treatment until the physician has approved continued use of the device.



More balance. More durable. More great solutions from OrthoAgility.

Email: info@QALmedical.com • Website: <http://www.QALmedical.com> • Phone: 1-888-430-1625 | +1-715-735-4727

Made in the USA • 3000 Woleske Road • Marinette, Wisconsin 54143

For more information contact your local representative.

Certifications:



Patent Numbers: 4,962,756; 5,115,806;
5,327,882; 5,697,892