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Instructions for use

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Therapeutic Table Stabilo Spidy with Stabilo System Cushion

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Thank you for choosing our product. NOEL Żebrowski sp. j. has designed and manufactured Therapeutic Table Stabulo Spidy with Stabulo System Cushion taking into consideration the highest safety and functional use standards. We hope that our product will meet your highest requirements and expectations. These instructions for use are an integral part of Therapeutic Table Stabulo Spidy with Therapy Stabulo System Cushion. Please read the instructions thoroughly before using the product.

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Therapeutic Table Stabulo Spidy with Stabulo System Cushion

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Stabulo Spidy Therapeutic Table, together with Vacuum Positioning Stabulo Therapy Cushion, is a solution for children with reduced muscle tone or/and muscle strength. Thus antigravity action, unburden upper limbs and allows them to activate. The satisfaction of completing a task with own hands is a stimulus for creative activity. Stabulo Spidy, allows children with reduced muscle tone to work independently with their hands and thus stimulates the child's energy and efficiency.

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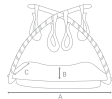
INDICATIONS

- reduced muscle tone and/or reduced muscle strength

Typ Type	Wymiary [cm] Dimensions [cm]		
	A	B	C
M	59	34	43

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LIMITATIONS

- strong spasticity and significant involuntary movements

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APPLICATIONS

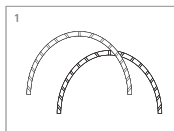
- increase the precision of movement
- support or impede the user's movement, in order to increase their activity
- learn a movement needed for everyday functioning, e.g. to operate a joystick in an electric wheelchair

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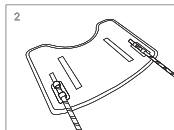
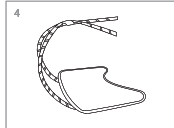
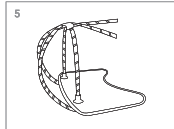
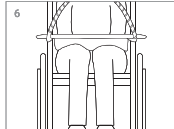
- increase the range of available movements, e.g. to activate the shoulder or to gain new possibilities for manual work
- support the manual functions of people with a progressive disease (e.g. muscular atrophy)

ATTENTION

- use only in the presence of a caretaker and according to the therapist's recommendations
- velcro on the tubes can stick to and shred clothes
- do not put hot meals or drinks on the table
- before use, the table should be permanently attached with fastening straps
- do not allow people who have not been properly instructed or trained before to use the table
- do not suspend any other elements that may burden the elements of the table
- be careful when bending the Stabilo Tubo - small bend radius may cause mechanical damage. The minimum bend radius is approx. 10 cm

1. Select two long Tubo and bend them into an arc.


2. Place the ends of the tubes in the holders under the table - the Tubo's length can be adjusted by moving the tubes under the table.

3. Attach the Stabilo cushion under the table.

4. Cross Tubo over the table top and then connect using the fastening strap.


5. If necessary, strengthen the structure - bend the shorter Tubo into an arc, attach the suction cups to the table (stick them to the table and tighten the suction cup by turning it to the right) place the ends of the tube in the suction cups, connect the Tubo to the previously installed tubes with a fastening strap.

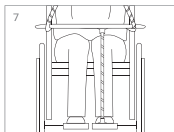6. On the opposite sides of the Stabilo cushion, attach the straps and attach the table to the armrests in the stroller.

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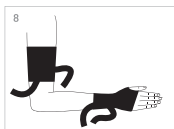
7. If fastening the table with straps to the armrests is insufficient, you can additionally stabilize it with a shorter Tubo - attach one suction cup to the front under the table, place the end of the Tubo in the suction cup, attach the second suction cup to the other end of Tubo and rest it (or fasten it) to a stable surface.

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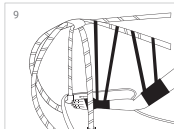8. Put the neoprene gloves on the child's hands and the neoprene armbands on the arms. Attach two straps to each glove and armband.

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9. Suspend the child's hand to the Tubo structure with the Velcro straps.

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MAINTENANCE

- regularly check condition of the product – pay attention to any cracks and rips
- screw on valve cap before washing
- hand wash cushion under running water using a standard detergent, e.g. a dish soap
- suitable for washing in a washing machine at 40°C cycle – fabric softener is sufficient, does not require use of washing powder
- wipe dry with paper towel
- standard disinfectants used in hospitals can be used
- do not use solvent-based cleaners such as toluene or acetone
- for Velcro products, Hoover the soft surface regularly to prevent loss of adherence

WARRANTY

- the warranty is provided for a period of 24 months
- if in doubt, please contact seller or manufacturer directly
- report any incidents to seller or manufacturer directly

NOEL Żebrowski sp. j.
ul. Christo Botewa 4A, 30-798 Kraków
+48 12 341 53 16
www.stabilo.krakow.pl
stabilo@stabilo.krakow.pl

Find Stabilo System Cushions on: